



Cold Velcro Pack Model#: 1905-715

**WARNING: Read and follow all directions carefully to avoid possible injury.**

Directions For Use:

1. Store in the freezer for at least 1.5 hours prior to use.
2. Once frozen, remove gel pack from storage pouch, place gel pack in a cloth or towel and place on affected area. **Always use product with a cloth or towel. Never apply directly to skin.**

Storage Instructions

Keep product stored in the resealable, reusable pouch in the freezer.

**CAUTION:**

1. Consult a physician prior to use.
2. People with poor circulation, diabetes, nerve damage, paralysis or sensitive skin should not use this product.
3. Although non-toxic, discard pouch if product leaks or is punctured.
4. Adult supervision is required for use on invalids or children. Do not use on infants (or children under the age of 2 years).
5. Do not use the pack in conjunction with liniments, sports creams, balms or lotions.
6. Do not use on broken skin.